



**PLEASE MAKE SURE YOU READ AND UNDERSTAND THE USER'S MANUAL.**




# CEPER<sup>®</sup>1

## RAPID USE IN 6 STEPS

CEPER Automatic CPR Chest Compression Device has been produced to contribute to the patient's life struggle in cases where continuous and effective manual CPR is not possible, when emergency personnel are insufficient or fatigued or when it is not possible to perform effective compressions when patient transport is required. See the User's Guide for the directions of use, indications, contraindications, warnings, precautions and potential adverse events.

**AUTOMATIC CPR CHEST COMPRESSION DEVICE**

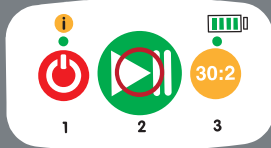

On / Off button.



Start / Stop compression button.



Start / Stop 30:2 mode button.



### Preparation and Application Steps of CEPER Automatic CPR Chest Compression Device

**1**

Perform Manual CPR while preparing CEPER Device.

**2**

Take the back plate out of the bag and make it ready for use.

**3**

Pause the manual CPR, hold the patient under the neck and under the arms, gently lift them up and position the back plate so that it aligns with the underarm. Make sure the back plate is positioned correctly relative to the sternum. The human figure drawn on the back plate will be a guide.

**4**

Take the CEPER out of its bag and align the massage piston with the middle of the lower half of the chest bone (sternum). Quickly attach the straps of the back plate and the buckle, stretch the straps to take up the device space.

**5**

Make sure the massage piston is in the middle of the lower half of the chest bone (sternum). Install the device battery. The device becomes ready for use with the BEEP sound. Attach it by stretching the shoulder belts in order to prevent the device from slipping.

**6**

When the battery is installed, the device becomes ready for use with a BEEP sound. When the device is on, if you press the START / STOP button (no 2), the piston moves back and forth after two BEEP sounds and starts continuous chest compression. If needed, the 30:2 ventilation cycle can be activated by pressing the button (no 3). To end CPR with the CEPER device, press the START/STOP button (no 2). After the BEEP sound, turn off the device by pressing the ON/OFF button (no 1). Detach the battery from the device.