

To request your FREE product sample, contact vernagel@vernacare.com



Carrying bodily fluids is no longer high risk.

A sachet of **VernaGel** solidifies up to 1.2 litres of bodily fluids in seconds, reducing the risk of spills and infection.

What is it?

VernaGel is a dry granule with rapid absorption properties. VernaGel stabilises bodily fluids and other medical waste in seconds, transforming them into an easily transportable, semi-solid gel.

Why VernaGel?



Prevents spillages & accidents



Helps reduce the spread of infection



Minimizes & eliminates residual odours



Improves patient and healthcare worker dignity



Reduces nurses workload



Saves laundry





VernaGel is a revolutionary product exclusively supplied by Vernacare and used worldwide.

How does VernaGel work?



1. Place one sealed VernaGel sachet into a container.



2. Add bodily fluid.



3. Within seconds, VernaGel will react and turn up to 1.2 litres of bodily fluids into a semi-solid gel.

Where can VernaGel be disposed?

- Disposal units
- Clinical waste
- Sharps bins

Where this is not possible, empty into a toilet or slop hopper.

Where can I use VernaGel?

VernaGel requires no special skills or training to use – making it ideal to use in almost every kind of healthcare situation where fluids are present.

- Urinals, bedpans, slipper pans, vomit bowls and washbowls of any material
- Sharps residues
- Spillages
- Liquid medicine disposal
- Colostomy bags
- Sanitary waste



DO NOT tip 'gelled' substance into wash basins or sinks.



DO NOT eat.

DO NOT leave patient unattended.



VernaGel sachets 100 sachets

Product Code: 450SC100 NPC Code: HFL8662



VernaGel Bottle Loose powder

Product Code: 456MA475 NPC Code: HFL8664



VernaGel Large TubLoose powder with scoop

Product Code: 456MA400



1 Western Avenue, Matrix Park, Chorley PR7 7NB, UK T: +44 (0)1204 529494 E: vernagel@vernacare.com www.vernagel.com