



LIFEVAC EUROPE LTD

- LifeVac is a patented **non-invasive**, portable airway clearance device.
- Interchangeable sized masks, clearly identified by colour coded rings.
- No risk of pushing the tongue or obstruction back in a panic situation.
- No risk of oral damage.
- Generates over 326mm Hg of suction, safely and effectively dislodging the obstruction.
- Can be used for full and partial obstructions.
- Saved hundreds of lives around the world from choking to death.
- **Only** airway clearance device with independent medical testing, peer reviewed medical publications, peer reviewed abstracts proving safety, effectiveness and lives saved.
- Comes in four different variations, Standard LifeVac Kit, LifeVac Travel Kit, EMS LifeVac Kit and Wall mounted LifeVac Kit.
- LifeVac is FDA, MHRA, UKCA, TGA, HPFB, MOH, SAPHRA registered/regulated as a class one medical device and CE marked.
- Can be applied if someone is standing, sitting or laying down. Can also be self applied in a choking emergency.
- Every kit comes with an easy to scan QR code which takes every end user to a free training video.

❖ Easy to hold handle for secure grip.

- Medically proven to be the worlds most effective, safest and easiest to use airway clearance device.
- Backed by multiple peer reviewed medical publications covering: Safety, effectiveness, lives saved, comparison studies and ease of use.
- Over 1,100 lives saved, with over 680 of these lives being children.
- Replaced free of charge if ever used in a choking emergency.

❖ Translucent bellows, makes it easy to identify if the obstruction enters this area.

❖ Patented one-way valve prevents any air being expelled through interchangeable sized masks.

Masks:

- ❖ Large Adult
- ❖ Medium Adult
- ❖ Small Adult/child
- ❖ Pediatric



❖ Interchangeable sized masks to fit a casualties facial features, as one size does not fit all.

Save A Life In 3 Simple Steps

Step 1

Place

Place LifeVac over the mouth and nose to create a seal.

Step 2

Push

The one-way valve prevents air from pushing food or objects downward when pressed.

Step 3

Pull

Then simply pull to create a one-way suction to remove the lodged food or object in seconds.



MHRA
Regulating Medicines and Medical Devices

**UK
CA**



FDA CE